




User Guide

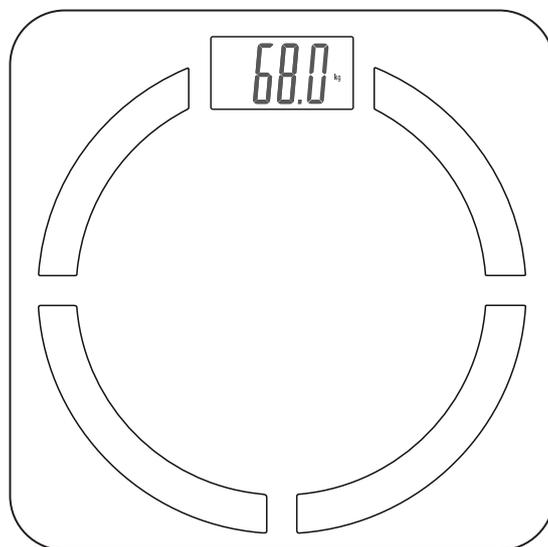

CF570BT

A Features Introduction

1. 8 parameters

-  Body fat
-  Body hydration
-  Body muscle
-  Body bone
-  Calorie
-  8 users
-  BMI
-  Visceral Fat

2. Weight will be synced into the phone and displayed when using it



3. Family member recognition



4. Health data analyze & health tips

In the health score page, tap on every measurement, the analysis and health tip will come out.

Health score




Weight & tips




BMI & tips




Body fat & tips

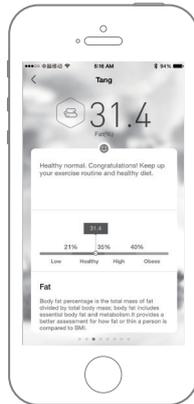


Body muscle & tips

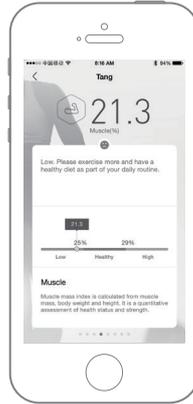

Visceral fat & tips

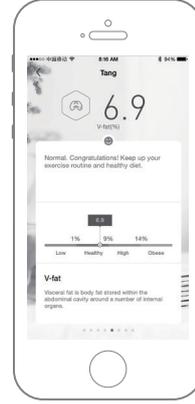

Body bone & tips

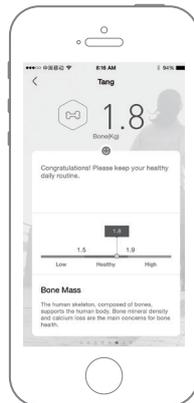

Body water & tips




BMR & tips



B Mobile & device readiness

1. Battery Installation

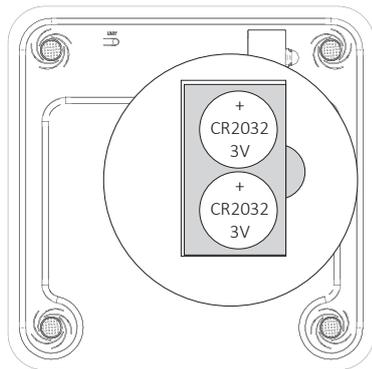

- 1 Open the battery door
- 2 Take out the insulation tape from battery compartment
- 3 close the battery door

2. Apply to Below Mobile Operating System



IOS 7.0



Android 4.3



Bluetooth 4.0

3. App Installation




- 1 Turn on your mobile phone WIFI or 3G or 4G
- 2 Scan the QR code aside to install the "OKOK" APP
Or download "OKOK" from APP store



C Registration

1. Getting started

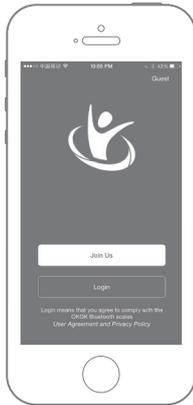


2. Email and password



3. Name



4. Gender

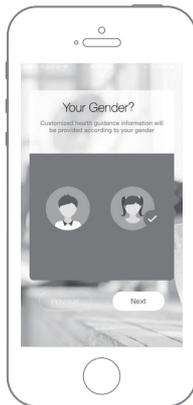


5. Birthday




6. Height



7. Select image

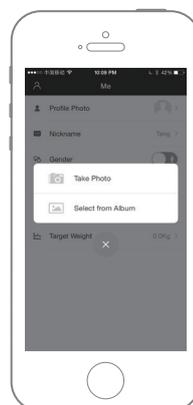

8. Target weight

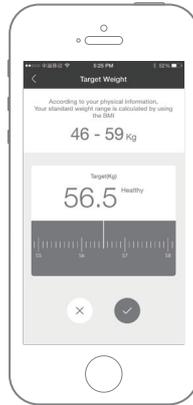

D Starting


1. Turn on bluetooth




2. Weighing start Stand on the scale



3. Binding and pairing

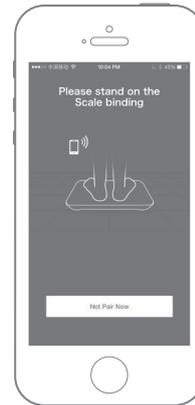

E Main page overview

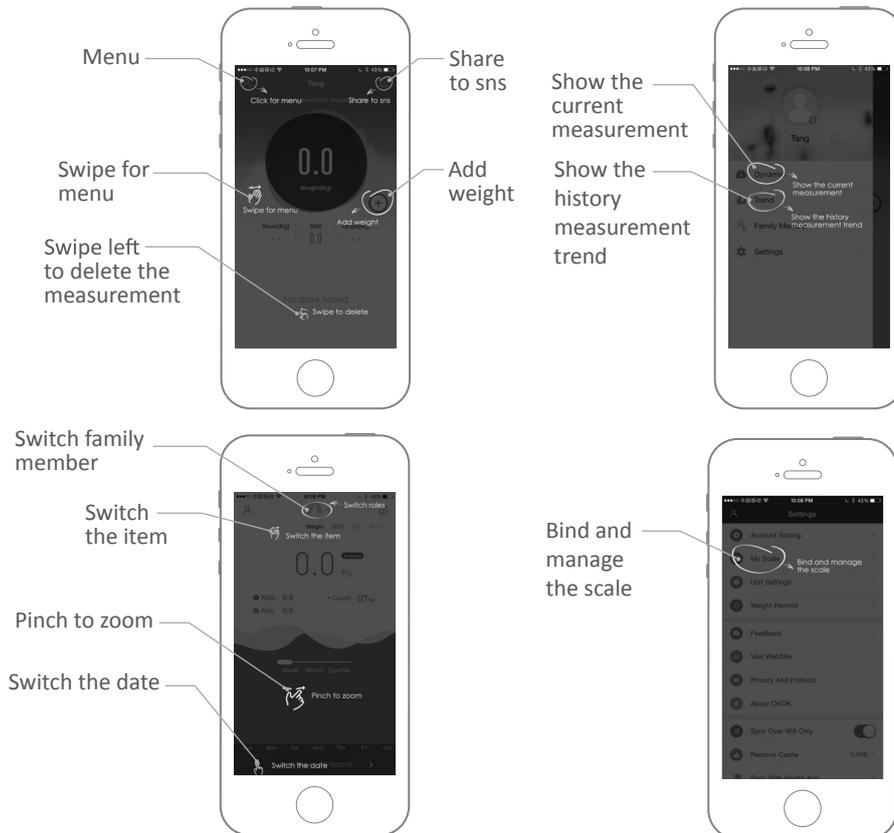

F Weighing & binding

1. First weighing, weight and BMI only, no other values




2. Second weighing, 7 measurement values




3. Binding

Follow to registration steps Or in the dynamic page, press the pair key, stand on the scale to finish the binding steps

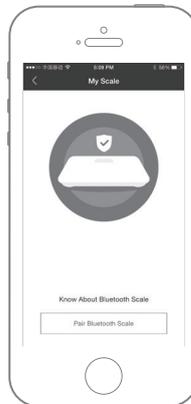

4. Unbinding

Enter into setting page, press my scale Press "unpaired" and "ok" to take place

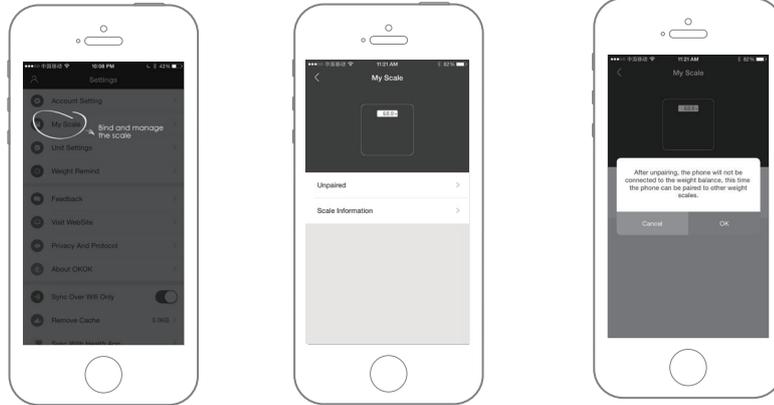


Reminder:


The scale only allow one person use at one time. Please unbind it to let another person to bind it.

G Family members adding and recognition

1. Enter into the family member page, press "+", then follow the same as registration steps

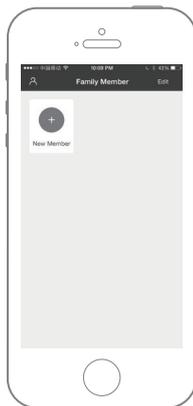

2. New weight also can apply for another family member, same as registration steps

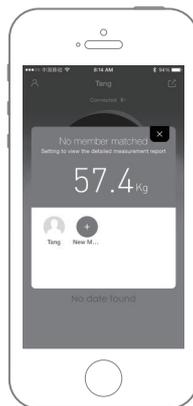


3. Total 8 members can be added, 3kg weight difference define the user automatically




H Modify password


1. Setting page

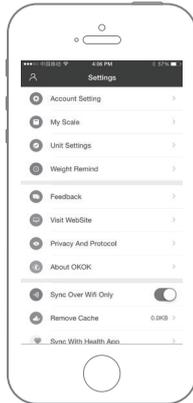

2. Account setting

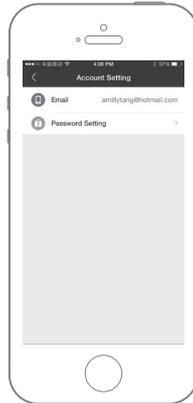

3. Password setting
press ok to finish

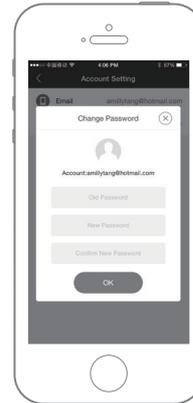

Reminder and Tips

1. Please make sure good contact between bare feet and the metal electrodes, otherwise it will display your weight only.
2. Ensure the scale is on a flat, even and firm surface. Avoid carpet or soft surfaces.
3. The surface of the scale will be slippery if wet. Please make sure both the glass and your feet are dry before use.
4. This scale is not water proof. Do not expose to moisture.
5. Clean the scale with a slightly damp cloth. DO not use any chemical/abrasive cleaning agents to clean the scale.
6. Please remove the batteries, if the scale is not to being used for domestic use within the home.
7. The appliance is intended solely for domestic use within the home.
8. Children should be supervised to ensure that they do not play with the appliance.
9. Before you use the appliance for the first time, please do as follows: carefully unpack the appliance and remove all the packaging material. Keep the material (plastic bag and cardboard) out of reach of children. Remove the batteries from the protective packaging and insert with the correct polarity.

